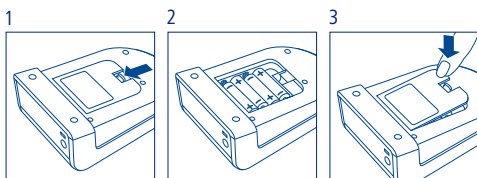
Tensoval® duo control

Guía rápida
Breves instruções
Σύντομες οδηγίες
Short instructions

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
GB After the batteries are inserted, the device automatically switches to the time function. The date (day indication) flashes in the display and can be changed by pressing the **+** (= +) or **-** (= -) buttons. Then store your settings by pressing the START / STOP button. Now the month indication flashes. Continue with setting the year and time in the same way.

- | | | | | |
|----------|----------|---------------|--------------------|----------------------|
| 1. Día | 2. Mes | 3. Año | 4. Hora | 5. Minutos |
| 1. Dia | 2. Mês | 3. Ano | 4. Horas | 5. Minutos |
| 1. Ημέρα | 2. Μήνας | 3. Χρονολογία | 4. Ώρα | 5. Λεπτά |
| 1. Day | 2. Month | 3. Year | 4. Number of hours | 5. Number of minutes |

2. Antes de realizar la medición · Antes da medição · Πριν από τη μέτρηση · Before measurement

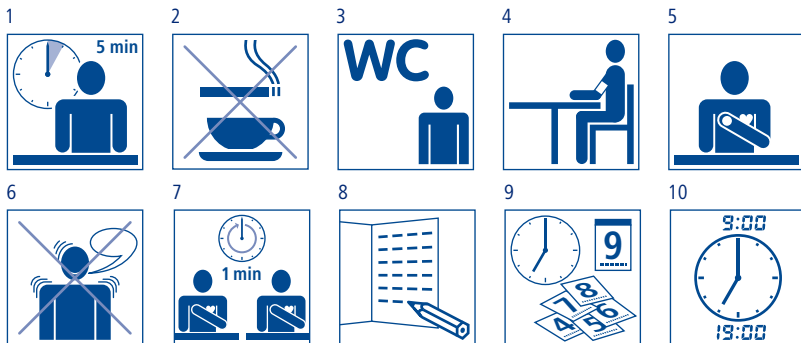2.1 Las 10 reglas de oro para la medición de la presión arterial · 10 regras de ouro para a medição da tensão arterial · Οι δέκα χρυσοί κανόνες για τη μέτρηση της αρτηριακής πίεσης 10 golden rules for blood pressure measurement

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GB

1. Rest before measurement.
2. Do not consume any coffee or nicotine before measurement.
3. Do not measure when you have a strong urge to urinate.
4. Take measurements whilst seated and from the naked upper arm.
5. Cuff has to be at the level of the heart.
6. Do not talk or move during the measuring procedure.
7. Wait at least one minute between two measurements.
8. Enter values in the blood pressure diary.
9. Take measurements regularly.
10. Always take measurements at the same time of day.

3. Proceso de medición · Procedimento de Mediçāo · Διαδικασία μέτρησης · Measuring procedure

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
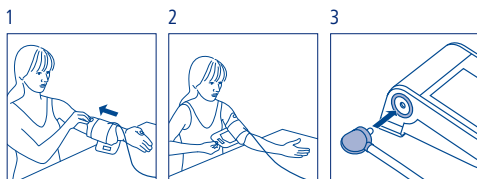
Measurement should be taken on the naked arm.

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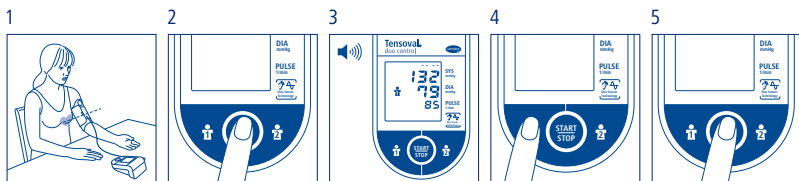If you wish to stop the measuring procedure during measurement, simply press the START / STOP button.

4. Después de la medición · Após a medição · Μετά τη μέτρηση · After the measuring procedure

4.1 Consulta de los valores de medición · Gravar os valores medidos · Ανάκληση των τιμών μέτρησης · Recalling the measured values

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Press the memory button corresponding to the person concerned while the device is in switched-off mode.

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Afterwards: individual values of the last 60 measurements with time and date.